ISOLATION AND FIXATION

HySolate Dental Dam

Upgrade your isolation



Upgrade Your Isolation

Complete isolation with dental dam is crucial in Endodontic and Restorative dentistry. It helps produce a dry working field to ensure both reliable and high-quality treatment results, protects patients from aspiration or ingestion of foreign objects, and protects soft tissue from contact with other instruments. Studies show that dental dam is also an effective infection control barrier and can significantly reduce the load of possible infectious airborne particles and minimize saliva and blood contaminated aerosol or spatter.^{1,2} For decades COLTENE has produced a comprehensive portfolio of high-quality dental dam, clamps, and accessories. Upgrade your Isolation with COLTENE.

COL



COMPREHENSIVE



SIMPLE

Dental dam application is actually a simple process. Choose the best possible material to meet your specific treatment needs as we guide you



SOLUTION DRIVEN

Our high quality products are engineered to facilitate isolation. Our products and training materials are the bridge that connect solutions with your treatment requirements.



- Peng, X., Xu, X., Li, Y. et al. Transmission routes of 2019-nCoV and controls in dental practice. Int J Oral Sci 12, 9 (2020). https://doi.org/10.1038/s41368-020-0075-9
 Cochran, M. et.al. The efficacy of the rubber dam as a barrier to the spread of microorganisms during dental treatment,

Benefits



Absolute dryness

DOVINE ONLY



Unblocked field of vision on the treatment area



Safety and protection



Selection Guidelines

Dental Dam

Our variety of dam materials ensure that the specific requirements of each clinical situation and treatment procedure are successfully and effectively met. Elasticity, the ability to retract, thickness, color and non-allergenic are relevant features to consider when choosing the most appropriate material.



Patient has anxiety or claustrophobia and rejects the application of Dental Dam.

1b 5b

20

DISPOSITION

actions to th

~

92

ALLERGY

Recommendations

RETRACTION

1a

Choose latex dental dam. Thicker dam allows for better retraction for a tight fit around the isolated area.

1b

Choose a more flexible and thinner dental dam with less retraction which will ease any stress on the gingiva.

ALLERGENS

TH CONS

2a

GINGIVAL STATUS

5

Choose powder-free, low protein dental dam to reduce the exposure to latex proteins and minimize the risk of developing a latex allergy.

2b

Choose a non-latex dam and avoid allergic reactions on latex proteins.

RESISTANCE



Choose latex dental dam as it affords the highest resistance to contact of foreign substances.



COLOUR OPTIONS

4a

Use psychology and education to take advantage of light and pleasent colours for a calming effect.

4b

Choose a darker dental dam to reduce eye fatigue and provide a good contrast to the working area.

SIZE AND APPLICATION AIDS

5a

Choose 127 x 127 mm dental dam for smaller patients.

5b

Choose Flexi Dam with built-in flexible frame for less stress on the patient.

GENERAL RECOMMENDATIONS

- Cut out the dental dam sheet so that nose breathing is possible
- Put Parotisroll on the upper lip to create space between nose and the dental dam sheet
- Use Dental Dam napkins to ease the contact of the dam material with the skin

HySolate Dental Dam Portfolio

Application



NON LATEX

Flexi Dam Non-Latex

HySolate Flexi Dam Non-Latex is made of an elastomer and performs with high elasticity while tension is moderate. Extra elastic and latex-free Dental Dam for soft retraction.

Flexi Dam Non-Latex Framed


HySolate Flexi Dam Framed is equipped with a built-inframe. You can get started without the need to have a frame at hand. The flexible frame is designed with working size of 100 x 105 mm to ensure for easy fitting.



	Flexi Dam Non-Latex	Flexi Dam Non-Latex Framed
Retraction	×	×
Elasticity		
Framed	-	\checkmark
Pre-printed	-	-
Thickness	medium	medium
Colours		
Scent	-	-
Children Size 127 × 127 mm	_	-

Stress-strain diagram

Flexi Dam Non-Latex performs with high elasticity while tension is moderate. It is ideal for softer retraction. **Dental Dam Non-Latex** is not as elastic as Flexi Dam while showing higher tension. Retraction forces on the gingiva are higher. It is the latexfree alternative to latex when seeking strong retraction. **Latex Dental Dam** provides best possible retraction.

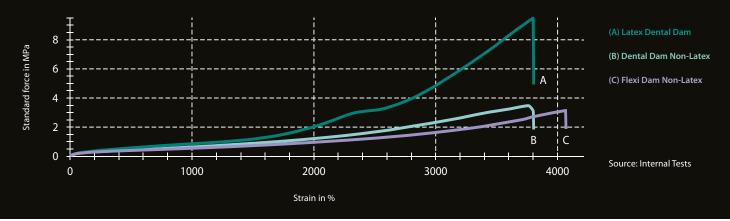

LATEX

Latex Dental Dam

HySolate Latex Dental Dam is made of pure, natural rubber latex and is powder free.¹ Powder free, low protein Dental Dam is a simple and clever measure to reduce the risk of developing latex hypersensitivity.

Latex Dental Dam BLACK EDITION pre-printed

The black Dental Dam has the template for marking the tooth position printed on. This saves the working step of marking the dam before punching. The black color provides ultimate, clear contrast and is ideal for taking photographs.

Fiesta Latex Dental Dam

Fruit scented powder free latex dam for greater patient and staff comfort. Each box contains an assortment of purple, blue and pink colored sheets.

¹ Powder content < 50µg/dm² Protein content < 2 mg/sheet</p>



Latex Dental Dam	Latex Dental Dam pre-printed	Fiesta Latex Dental Dam
****	××××	××××
H	⊢ I	⊢ I
-	_	_
-	~	-
medium	medium	medium
_	_	\checkmark
\checkmark	-	\checkmark


Order Information

HySolate Flexi Dam Non-Latex

SIZE	COLOUR	THICKNESS	CONTENT
		medium	Sheets
152 × 152	Purple	H09945	30
	ALL STATE AND A STATE OF A DATE OF A		

HySolate Flexi Dam Non-Latex Framed

	A 1998年1月1日日日日本市内市市市市市市市市市市市市市市市市市市市市市市市市市市市市市市市市		
SIZE	COLOUR	THICKNESS	CONTENT
		medium	Sheets
100 × 105	Purple Framed	H00750	20
Parks:351.00 2522		表示 中国社会社会社会社会社会社会社会社会社会社会社会社会社会社会社会社会社会社会社会	

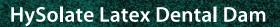

SIZE	COLOUR	THICKNESS	CONTENT		
		medium	Sheets		
	Green	60034419	36		
152 × 152	Black pre-printed	60031352	36		
	Blue	60034442	36		
127 × 127	Green	60034415	52		
12/ × 12/	Light	60034425	52		

HySolate Fiesta Latex Dental Dam

SIZE	COLOUR	THICKNESS	CONTENT
		medium	Sheets
152 × 152	Fiesta	60034436	36
127 × 127	Fiesta	60034435	52



COLTER

Dental Dam Kits

*Sheets are subject to customization and inquiry on availability

*Sheets are subje	ect to customization and inquiry on availability		
HO2790	Wingless Complete Kit Contents: HYSOLATE DAM GREEN MEDIUM 6" x 6" (152 x 152mm) 36 Sheets 6" (152mm) Plastic Frame (H01414) Fiesta 9W Wingless Color Coded Clamp Pak 6" Template Wedjets® Cord (Small) Dental Dam Punch Dental Dam Forceps	HO2790 IN STI	DENTAL DAM KIT WINGLESS Starter Kit Contents: HYSOLATE DAM GREEN MEDIUM 6"x6" (152 x152mm) 36 Sheets 6" (152mm) Metal U shaped Frame Fiesta 9 Wingless Color Coded Clamp Set 6" Template Wedjets® Cord (Yellow) Pouch -2 mtrs COLTENE INDIA Punch COLTENE INDIA Forcep
HO2778	Winged Complete Kit Contents: HYSOLATE DAM GREEN MEDIUM 6" x 6" (152 x 152mm) 36 Sheets 6" (152mm) Plastic Frame (H01414) Fiesta 9 Winged Color Coded Clamp Pak 6"Template Wedjets [®] Cord (Small) Dental Dam Punch Dental Dam Forceps	HO2778 IN ST	DENTAL DAM KIT WINGED Starter Kit Contents: HYSOLATE DAM GREEN MEDIUM 6"x6" (152 x152mm) 36 Sheets 6" (152mm) Metal U shaped Frame Fiesta 9 Winged Color Coded Clamp Set 6" Template Wedjets [®] Cord (Yellow) Pouch -2 mtrs COLTENE INDIA Punch COLTENE INDIA Forcep
HO9715IN	Dental Dam Pedo Kit Contents: Dental Dam Punch Dental Dam Forcep 5" Dental Dam Frame Brinker Clamp Set 5" TEMPLATE HYSOLATE READY CUT DAM GREEN 5" x 5" (127 x 127mm) 52 Sheets Wedjets [®] Cord (Small)	HO9715INST	Dental Dam Kit PEDO Starter Kit Contents: Dental Dam Punch Dental Dam Forcep 5" Dental Dam Frame Brinker Clamp Set 5" TEMPLATE HYSOLATE READY CUT DAM GREEN 5" x 5" (127 x 127mm) 52 Sheets Wedjets [®] Cord (Yellow) Pouch -2 mtrs
between 1,0	tyle stainless steel punch with 5 holes and 2,3 mm in diameter. The Ainsworth be hinge-joint behind the die plate.		#5-Anchor Tooth
Forceps REF H01262 The light-we	ight forceps with a comfortable grip facilitates		#3-Bicuspids & Canines #2-Maxillary Anterior Incisors

the secure transfer and fixation of Dental Dam clamps to the tooth.

#1-Mandibular Anterior Incisors

Fixation

Clamps


Dental dam fit partly depends on how it is secured to the teeth. COLTENE makes fixation simple with both Wingless and Winged clamps. Winged clamps have projections at the jaws while wingless do not. Our aim is to accommodate application techniques with any dentist's style. Our dental dam clamps are manufactured from high quality stainless steel for greater durability.



Wingless clamps are positioned intra-orally first and then the dam is stretched over the clamp and tooth. Our wingless clamps are designated with a W for easy identification. Winged clamps combine steps by placing the dam, clamp and frame simultaneously. Once applied, the dam is stretched off the wings of the clamp.

Four-Point Contact

Select a clamp that will maintain four-point contact with the tooth's proximal surfaces. The clamp is positioned with the bow on the distal of the tooth. Universal clamps can be used on the left and the right side of both the upper and the lower jaw. Both clamp jaws are equal.

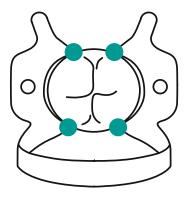

Wedjets – Stabilization Cord 5

Wedjets elastic stabilizing cord provides a quick and easy way to retain dental dam and can complement or supplement clamp usage.

- Standard clamps
 Fiesta colour coded clamps
 Tiger clamps
- 4 Brinker clamps
- 5 Wedjets

Brinker Clamps 4

HySolate Wedjets

ND

LATEX DENTAL DAM STABUEING CORD 7 II. (21m) contrivuous length


C & / I WE ONLY CE

#COLTENE

4

5

Extra inclined beaks provide maximum retraction without laceration of gingival tissues

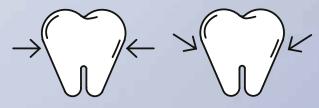



Retention – apical effect

3

Select your clamps

Select a clamp that will maintain four-point contact with the tooth's proximal surfaces. The clamp is positioned with the bow on the distal of the tooth. Universal clamps can be used on the left and the right side of both the upper and the lower jaw. Both clamp jaws are equal. Our comprehensive selection of anatomical shapes help aid in quick and easy placement.

Standard clamps

Gloss finished standard clamps.





#W00 Premolar

#00 Premolar

Fiesta clamps

Each clamp is individually color coded for unmistakable identity. Experienced and novice associates will appreciate this time saving identification method when using Fiesta Color Coded Clamps.





#W9

Anterior






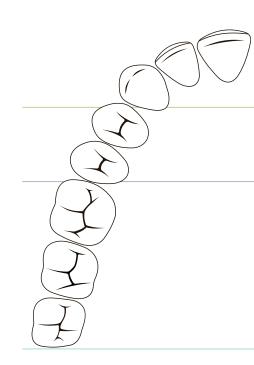

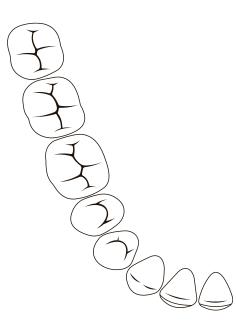

	ANTERIOR SIZES						PREMOLAR SIZES							
Clan	nps	#9	#212	#B4	#B5	#B6	#00	#1	#2	#2A	#B4	#B5	#B6	
σ	Standard	×					×	×	×	×				
Winged	Fiesta	×							×	×				
5	Tiger													
SS	Standard	×	×				×		×	×				
Wingless	Fiesta	×							×	×				
N	Brinker			×	×	×					×	×	×	

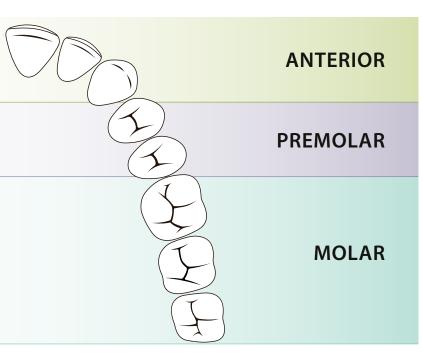

Tiger clamps

Clamps with serrated jaws to be used for structurally compromised tooth.





#12A

Molar Lower jaw right Upper jaw left

Molar Lower jaw left Upper jaw right

Brinker clamps

Brinker Clamps have extra inclined beaks to provide maximum retraction of the gingiva. They provide maximum retraction without laceration of gingival tissues.



#B1 Molar Lower jaw right



#B2 Molar Upper jaw left

MOLAR SIZES												
#3	#4	#7	#8	#8A	#12A	#13A	#14	#14A	#56	#B1	#B2	#B3
×	×	×	×	×			×	×				
	×	×		×	×	×		×				
					×	×						
×		×	×	×			×	×	×			
×		×	×	×				×	×			
										×	×	×

Order Information

WINGLESS



PREMOLAR

MOLAR

Standard | #W9 H02770



H07792







Brinker | #B6

H01053



Standard | #W00 H02769



Standard | #W7

H02765

Fiesta | #W3

H02796

Brinker | #B1

H01048

Standard | #W2 H02768



Standard | #W2A H02775



H02794



Fiesta | #W2A H02795





H03814







Fiesta | #W56 H02803



. H02767





Brinker | #B2 . H01049



Standard | #W8A H02766



Brinker | #B3 . H01050



Fiesta | #W8A

H02799



Fiesta | #W14A . H02802

Standard | #W14A H02764





CLAMP SETS

H02801 H	YGENIC FIEST	A 9W Colour Coded Wingless Clamp Pak
H09966 H	YGENIC FIEST/	A 9 Colour Coded Winged Clamp Pak
H09956 H	YGENIC FIEST	A Colour Coded Clamp Organizer
H02701 H	YGENIC Gloss	Finish System-9 Winged Clamp Pak
H01054 H	YGENIC Brinke	er Clamp Set

WINGED







Standard | #1

H02703



Standard | #2 H02761



Standard | #2A H05688



Fiesta | #2 H09957





H05689



Standard | #14A H02757





Standard | #4 H02704



Tiger | #12A H02706



Fiesta | #13A H09964

Standard | #8 H02760

Fiesta | #4 H09959



Standard | #7

H02758

Tiger | #13A H02705



H09965



Standard | #8A H02759



Fiesta | #7 H09960



Standard | #14 H05690



Fiesta | #8A H09961



can complement or supplement clamp usage.







© COLTENE – www.coltene.com

Coltène/Whaledent AG Feldwiesenstr. 20 9450 Altstätten/Switzerland T +41 71 757 53 00 F +41 71 757 53 01 info.ch@coltene.com Coltene Whaledent Pvt. Ltd. (A Subsidiary of Coltene/Whaledent AG, Switzerland) (ISO 13485:2016) Plot No. 124, Jawahar Co-op Industrial Estate Limited, Kamothe, Navi Mumbai, Dist. Raigad, Maharashtra-410206 Tel: 022-2740 1841 | E: InfoIndia@coltene.com www.coltene.co.in

#COLTENE